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Produkbeschreibung

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EN Instructions for use

Product description

Grossolind Neutral dressings are non-medicated, sterile, paraffin gauze dressings for wound treatment.

Composition

Leno weave (tulle) cotton gauze fabric impregnated with an anhydrous ointment consisting of:

- White vaseline
- Diglycerol ester of mono- and dicarboxylic fatty acids
- Microcrystalline hydrocarbon wax

Properties and mode of action

The tulle's open weave design allows exudate to pass into a layer of absorbent dressing placed on top of it, so helping to avoid maceration. The ointment keeps the edges of the wound supple, promotes the formation of granulation tissue and epithelium and prevents the dressing from adhering to the wound. Dressings can be changed painlessly without irritating granulation tissue and new epithelium.

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Kontraindikations

Do not use Grossolind Neutral on patients who may be allergic to any of its ingredients.

Product disposal

In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable and local laws, regulations and infection prevention standards.

Special precautions

In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary on these population groups, this dressing should be used with caution following a clinician's recommendation.

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

Before opening the product packaging, inspect it carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.

Incident reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on medical devices), if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

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FR – Laboratoires PAUL HARTMANN S.A.s.l. - Châtenois – 67607 Sélestat CEDEX