medela

Hydrogel Pads

Instructions for use

The Hydrogel Pads may be used in the treatment of superficial pressure sores, wounds and can be used as soothing pads to reduce local pain and; such as discomfort for sore, cracked, painful nipples by breastfeeding mothers.

Instructions for use

- Wash hands before using the product.
- Visually inspect sterile packaging of the device for damage before opening. Devices with a damaged packaging system must not be used.
- Open sterile pouch and remove Hydrogel Pad from the plastic liner.
- Avoid touching the gel side with your fingers.
- Gently place the gel side of the pad directly over your nipple.
- Prior to breastfeeding, remove the Hydrogel Pad and place it on a clean surface with the gel side facing up. Gently wash the breast with warm water and pat dry. Then, start breastfeeding.
- After breastfeeding, reapply the Hydrogel Pad.
- For more cooling sensation, unopened Hydrogel Pads can be placed in the fridge shortly before use.
- Do not use soaps or detergents to clean pad.
- Store Hydrogel Pads in a cool, dry place.
- The pad can be used up to 24 hours.
- Hydrogel Pad becomes dry or milky, please discard and use a new one.
- Dispose as per local regulation.

Important Safeguards

- Use only as indicated.
- Keep out of reach of children, especially the plastic liner.
- For external use only.
- Extra care should be taken if using with cracked nipples or potential infection.
- If nipples are bleeding or have a burning sensation, contact your healthcare professional.
- To prevent nipple pain, it is important that baby feeds at mother's breast in a correct position.
- Use of the device beyond the recommended time period may cause contamination.
- Reprocessing or cleaning could cause loss of device function.
- Do not use in case of known allergies to any of the ingredients. The product may contain traces of known allergen Polycaprylamide.
- If you have a known allergy to this substance it is recommended not to use the Hydrogel Pads.

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UK Responsible Person
Lotus Global Co. Ltd.
23 Maine Street, Reading, RG2 6AC, England, United Kingdom

UK Importer
Medela UK Ltd.
Unit 3, Huntsman Drive, Northbank Industrial Park, Irlam, Manchester M44 5EG, United Kingdom
www.medela.com/support

CE
0123
MD
Zhejiang Longterm Medical Technology Co., Ltd.
ShuangShan Road 277, Fuxi Street, Deqing County, 313200 Huzhou City, Zhejiang Province, People's Republic of China
www.longterm-health.com
innomed@longterm-health.com

CE
0123
MD
Zhejiang Longterm Medical Technology Co., Ltd.
ShuangShan Road 277, Fuxi Street, Deqing County, 313200 Huzhou City, Zhejiang Province, People's Republic of China
www.longterm-health.com
innomed@longterm-health.com

MCAG Ref. HGP IFU - 11CE/V1.0
© Medela AG www.medela.com

ENGLISH
 Hydrogel Pads Instructions for use Intended Use / Purpose The Hydrogel Pads may be used in the treatment of superficial pressure sores, wounds and can be used as soothing pads to reduce local pain and; such as discomfort for sore, cracked, painful nipples by breastfeeding mothers. Instructions for use - Wash hands before using the product. - Visually inspect sterile packaging of the device for damage before opening. Devices with a damaged packaging system must not be used. - Open sterile pouch and remove Hydrogel Pad from the plastic liner. - Avoid touching the gel side with your fingers. - Gently place the gel side of the pad directly over your nipple. - Prior to breastfeeding, remove the Hydrogel Pad and place it on a clean surface with the gel side facing up. Gently wash the breast with warm water and pat dry. Then, start breastfeeding. - After breastfeeding, reapply the Hydrogel Pad. - For more cooling sensation, unopened Hydrogel Pads can be placed in the fridge shortly before use. - Do not use soaps or detergents to clean pad. - Store Hydrogel Pads in a cool, dry place. - The pad can be used up to 24 hours. - Hydrogel Pad becomes dry or milky, please discard and use a new one. - Dispose as per local regulation. Important Safeguards - Use only as indicated. - Keep out of reach of children, especially the plastic liner. - For external use only. - Extra care should be taken if using with cracked nipples or potential infection. - If nipples are bleeding or have a burning sensation, contact your healthcare professional. - To prevent nipple pain, it is important that baby feeds at mother's breast in a correct position. - Use of the device beyond the recommended time period may cause contamination. - Reprocessing or cleaning could cause loss of device function. - Do not use in case of known allergies to any of the ingredients. The product may contain traces of known allergen Polycaprylamide. - If you have a known allergy to this substance it is recommended not to use the Hydrogel Pads.

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Contraindications: The Hydrogel Pad is not indicated for full-thickness wounds, heavily bleeding wounds, full thickness burns or as a covering for deep wounds.

Ingredients: Water, glycerol, polymers

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Transport and storage in dry conditions and at room temperature.
Store in normal room temperature for the 3 years shelf life.
Sterile as long as pouch is not damaged.

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