## CUTICERIN®

Mažai lipantits tulle gras tvarstis

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Smith & Nephew Medical Limited, 101 Henslie Road, Hull HU3 2BN England
©Trade Marks of Smith-Nephew. ©Smith-Nephew 2024
Printed in / Imprimé en: Canada
Date of issue 03/2024. Revision 01
www.smith-nephew.com
This product may be covered by one or more US Patents
www.smith-nephew.com/patents

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## CUTICERIN®

Low adherent tulle gras dressing

**Description**
CUTICERIN® Dressings are made of a smooth acetate fabric with a fine mesh and water repellent ointment coating.

**Mode of action**
CUTICERIN Dressings act as a wound contact layer between the wound and a secondary dressing. The open weave structure allow exudate to pass into an absorbent secondary dressing e.g. MELOLIN®. Dressings are coated with a low adherent ointment to minimise pain during dressing change and removal.

**Indications**
Management of superficial exuding wounds such as:

- Burns
- Radiation injuries
- Abrasions
- Split-thickness skin graft donor sites.

**Contraindications**
Do not use on patients with known allergies to any of the components:

- Petrolatum
- Parafin
- Wool wax alcohol.

**Precautions**
Seek medical advice immediately for all serious wounds and burns or if redness or discomfort occurs. If in doubt, seek medical advice regarding severity of the wound. Should signs of sensitivity or irritation occur, discontinue use and seek medical advice. CUTICERIN Dressings are a single use only product. Device should not be re-used. Cross-contamination can occur and the therapy may be ineffective. Opening the dressing pouch compromises the sterile barrier, therefore any unused dressing should not be retained for application at a later date.

**Avoid contact with eyes** as may cause irritation.
**Fire Hazard:** Do not smoke or go near naked flames; clothing and bedding with this product dried on them can catch fire easily

Product Availability			
Code	Size	Availability	
66045560	7.5 cm x 7.5cm / 3in. x 3in.	Carton of 50	
66045561	7.5 cm x 20cm / 3in. x 8in.	Carton of 50	
66045562	7.5cm x 7.5cm / 3in. x 3in.	Carton of 10	
66045563	7.5cm x 20cm / 3in. x 8in.	Carton of 10	
66045564	10cm x 10cm / 4in. x 4in.	Carton of 10	
66045502	20cm x 40cm / 8in. x 16in.	Carton of 25	
66045503	7.5cm x 20cm / 3in. x 8in.	Carton of 25 x 3	

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**FIRE HAZARD**
Do not smoke or go near naked flames; clothing and bedding with this product dried on them can catch fire easily

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