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Dear User, Please read the instructions for use carefully before using EVOLSIN® SCAR PLASTER, because they contain important information. If you have any questions, please contact your doctor or pharmacy.

Evolsin® Scar Plaster

EVOLSIN® SCAR PLASTER are a medical device for self-application for the simple, safe, and effective reduction of the visibility of new and old scars.

Purpose

Reduce the visibility of hypertrophic (enlarged), raised, and reddened keloid scars (protruding scars) and increases the moisture of the skin. For scars following surgery, injuries, and Caesarean sections, as well as hypertrophic, acne, and burn scars

Indications

The EVOLSIN® SCAR PLASTER has been specially developed for old and new scars in order to noticeably reduce their visibility and increase the moisture of the skin. It is a clinically and dermatologically tested product and can be used on hypertrophic (enlarged), raised, and reddened keloid scars (protruding scars). The EVOLSIN® SCAR PLASTER was designed to build a semi-occlusive (semi-permeable) barrier that improves the moisture concentration of the scar tissue while also increasing the temperature of the scar tissue and providing protection. This is intended to activate the skin's own regeneration process and support the remodelling of the scar. Scars are expected to become flatter, lighter, and softer, and thus less visible. Clinical studies have shown positive results after 8 weeks of consecutive use for at least 12 h a day. Initial results can be seen after just 3 to 4 weeks. Using the patch for longer than 8 weeks may further improve the appearance of the scar. In individual cases, it can take several months for the scar to noticeably reduce. Results may vary from person to person and from scar to scar.

Product information

- For scars following surgery, injuries, and Caesarean sections, as well as hypertrophic, acne, and burn scars
- Repositionable/can be moved
- Protects the scar
- Waterproof
- Plaster can be cut to size
- Dermatologically tested
- Clinically proven
- Easy to use
- Suitable for sensitive skin

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Patient and user group

For self-application by people aged 3 and over. For children, only use as directed by a doctor and under the supervision of an adult.

Contraindications/warnings/side effects

Keep the product out of the reach of children to prevent the risk of swallowing and suffocation. If swallowed, seek medical advice immediately and present the packaging and/or package insert and plaster.

Any serious incidents that occur in relation to the product must be reported to the manufacturer and the relevant authority in the EU Member State in which you live.

- This product must not be used:
- on open wounds or acute burns
 - if scabbing is still present
 - on inflamed areas of skin in the event of any dermatological disease (e.g. psoriasis, herpes, acne, etc.)
 - for external use only or not for mucous membranes
 - in the event of known allergies to any of the ingredients
 - do not use in combination with ointments or creams
- In rare cases, skin irritation or a rash may occur with plasters during use. In this case, stop using them and consult a doctor.

EVOLSIN® SCAR PLASTER Plasters are intended for single use only. This means that you should not reuse plasters that have been cut to size after using them once. Dispose of them after use.

Application/How does the Evolsin® Scar Plaster work?

In the event of a surgical procedure, application can be started as soon as the wound is completely closed and healed and after the sutures/staples have been removed. The same applies to acne or burn scars.

- 1 Clean and dry the affected area of skin/scar area.
- 2 Removing any hair in the area of the scar can increase adhesion and durability.
- 3 Cut the plaster to a suitable size.
- 4 Remove the protective film and apply the plaster to the skin so that the scar is completely covered. If necessary, you can apply several plasters next to each other.
- 5 Apply the plaster to the desired area and wait a few minutes until it is perfectly in place. If necessary, EVOLSIN® SCAR PLASTER can be secured using a fixing plaster or elastic bandage.
- 6 If possible, wear the plaster for at least 12 hours a day. It is recommended to gradually increase the duration of use and to change the plaster daily, as well as if it becomes dirty or does not adhere well.

In the following days/weeks/months:

When performing personal hygiene, be careful not to accidentally dislodge or move the EVOLSIN® SCAR PLASTER.

Storage instructions

Keep these instructions up until the product has been completely used up. Store the EVOLSIN® SCAR PLASTER in its original packaging in a dry place at normal room temperature. Store away from direct heat/sunlight. The expiry date and batch number are printed on the bottom of the folding box. Do not use after the expiry date.

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