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GB
SANOHRA *pro sleep* – Instructions

Sizes:
S
ø 7 – 9 mm,
M
ø 8 – 11 mm,
L
ø 9 – 13 mm

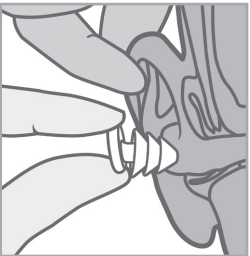
Intended use
Reusable ear plugs against harmful noise (EN352-2:2020, EU 2016/425). Testing laboratory for conformity assessment and surveillance according to module C2: PZT GmbH (1974), Bismarckstraße 264B, 26389 Wilhelmshaven, Germany. The product must only be used as intended.

Note: the plugs must be fitted, adjusted and maintained in accordance with the manufacturer’s instructions.

SANOHRA <i>pro sleep</i> – for quiet nights and productive days

Experience restful sleep with **SANOHRA *pro sleep*** earplugs. These innovative earplugs offer optimum noise reduction by effectively muffling disturbing noises such as traffic noise and snoring. Thanks to their ergonomic, oval shape, they adapt perfectly to the natural ear structure for maximum comfort. Improve your sleep quality and enjoy deep, undisturbed sleep. Start every day refreshed and full of energy.

- ✓ **Optimum noise reduction**
- ✓ **Improved sleep quality**
- ✓ **Comfortable fit thanks to oval shape**

Using SANOHRA *pro sleep*
Take your arm over the head and slightly pull up the upper part of the auricle of the ear on the other side of the head. The ear canal is now widened. Hold **SANOHRA *pro sleep*** in the shaft between the thumb and index finger of the other hand and insert **SANOHRA *pro sleep*** into the ear canal with light circular movements with the lamellae in front. Proceed in the same manner with the other ear.

Removing SANOHRA *pro sleep*
Before removing slightly lift the auricle as described. **Do not pull out the ear protection** suddenly but remove slowly with light circular movements.

Children should use the ear protection only under the supervision of adults. For children under the age of 3 years, please ensure that the ear protection is inserted and removed only by adults. The use of ear protection should be discussed earlier with your doctor.

WARNING! RISK OF CHOKING! SMALL PARTS. Children should use the ear protection only under the supervision of adults. The earplugs should be assembled, adjusted and maintained in accordance with the manufacturer’s instructions.

Warnings when using SANOHRA *pro sleep*
Caution: If the usage instructions and instructions for inserting are not followed, the protection effect of **SANOHRA *pro sleep*** against noise can be significantly affected. Earplugs should be worn constantly in noisy areas.

Warning: Sudden or rapid removal of the earplugs from the ear canal can damage the eardrum.

SANOHRA *pro sleep* can be reused if proper care is taken. The product can be cleaned using mild detergents (soap). Check the product for damages at regular intervals and replace if necessary. This product can be adversely affected by certain chemical substances. Please ask the manufacturer for additional information. It is advisable to store **SANOHRA *pro sleep*** after use again in the small box provided. It should be stored in a cool and dry location. All the parts of the earplug are designed and manufactured such that they do not cause any injuries when worn as described in the usage instructions.

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Conforms to European Regulation (EU) 2016/425 for personal protective equipment (CE), EN352-2:2020, EN24869-1. Notified Body PZT GmbH Number: 1974

Declaration: Innosan GmbH hereby declares that the PPE SANOHRA pro sleep conforms to Regulation (EU)2016/425. The full text of the EU Declaration of Conformity is available at the following Internet address: <https://www.sanohra.com/downloads>

CE 1974
Manufacturer: Innosan GmbH · D-68723 Schwetzingen · Beethovenstraße 3 · Germany
E-Mail: info@sanohra.com · Internet: www.sanohra.com

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Üretici: Innosan GmbH · D-68723 Schwetzingen · Beethovenstraße 3 · Germany
Producent: Innosan GmbH · D-68723 Schwetzingen · Beethovenstraße 3 · Niemcy