OROXID® sensitiv - oralna raztopina ali oralno pršilo

1. Opis izdelka in namen uporabe

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Proizvajalec

OROXID® sensitiv - oral spray or oral solution

1. Product description and intended use

OROXID® sensitiv is used for maintaining oral hygiene, healthy gums and to help against inflammatory conditions of the oral cavity. It is also intended for persons who are prone to the formation of inflammatory changes in the oral cavity (smokers, patients with diabetes, individuals with weakened immune systems, etc.).

OROXID® sensitiv, due to its mechanical properties and the release of oxygen in the form of mild foaming, positively affects the microenvironment of the damaged mucosa. Therefore, it is recommended as a complementary oral hygiene for fixed dental appliances, partial dentures, fixed prosthetic substitutes (crowns, bridges) and implants. It improves the results of oral hygiene for patients undergoing orthodontic irregularities which interfere with normal dental hygiene. Hydrogen peroxide can help in preventing the formation of plaque and tooth discoloration. It also helps to restore fresh breath and reduce deposits on the tongue and oral mucosa, usually induced by the presence of fungal and bacteria pathogens.

OROXID® sensitiv acts only on the surface, the hydrogen peroxide does not penetrate into deeper tissues.

As an excipient **OROXID® sensitiv** also contains Actipone® PX3 - an extract of herbs, has a soothing effect on irritated and sensitive tissues in the oral cavity.

The product is available in the form of a 50 ml, 100 ml spray and a 30 ml, 150 ml and 250 ml solution pack.

2. Administration

Use **OROXID® sensitiv** two to three times a day, preferably during or after brushing your teeth.

OROXID® sensitiv spray: spray the solution into the oral cavity with 3 to 4 pumps of the spray, swirl it around your mouth for about 30 seconds and spit it out.

OROXID® sensitiv oral solution: pour approximately 10 ml of liquid (content of two caps) in to a glass, swirl it around your mouth for about 30 seconds and spit it out.

OROXID® sensitiv contains a low concentration of hydrogen peroxide, and is therefore suitable for prolonged use, and in humans with sensitive mucosa. **OROXID® sensitiv** is not intended to treat serious inflammatory diseases of the oral cavity, covering the deeper tissues. In this case, consult your case with a doctor. For preventive purposes and the maintenance of good hygiene of the oral cavity the product should be used once or twice a day in a manner as described above. The dispersion of the solution in the mouth can be a felt as foam, this indicates the release of active oxygen.

3. Side effects and precautions

When applied to the mucosa, hydrogen peroxide at a low concentration rarely causes a feeling of irritation. In the case of a hypersensitive reaction or irritation discontinue use of the product. The feeling of irritation is transitory in nature and stops after a few minutes or after washing with water. Use of an oral solution **OROXID® sensitiv** is safe and does not damage the hard and soft tissues of the oral cavity and does not increase the risk of long-term side effects.

OROXID® sensitiv is only intended for an application in the oral cavity. **OROXID® sensitiv** is not intended to be consumed. Avoid contact with eyes.

Contact with eyes

Contact of **OROXID® sensitiv** with the eyes may cause irritation, tearing and blurred vision, but serious injuries are unlikely. After contact with the eyes rinse the affected eye with tap water or saline solution and, if necessary, visit your doctor.

Ingestion

When **OROXID® sensitiv** is ingested it can cause mild irritation of the lining of the gastrointestinal tract, vomiting and diarrhea. When the solution is ingested in large quantities or in the occurrence of severe symptoms contact your doctor.

Inhalation

OROXID® sensitiv at room temperature does not produce significant quantities of hydrogen peroxide vapors. When heated, dispersion or solution can lead to acute inhalation of hydrogen peroxide, which can cause irritation of the nose, throat and respiratory tract.

Spillage

Do not open the container. If spills occur, and the solution comes into contact with skin, wash it off with soap and water. Wipe spilled solution with a sufficient amount of paper and then wash your hands.

4. Ingredients

OROXID® sensitiv contains deionized water, 1.5 % hydrogen peroxide solution, Actipone® PX3 (Aqua, Glycerin, PEG-40 Hydrogenated Castor Oil, Zingiber Officinale Root Extract, Commiphora Myrrha Resin Extract, Echinacea Purpurea Extract), flavors and stabilizer.

5. Storage and shelf life

Store **OROXID® sensitiv** at room temperature of 5 °C to 25 °C.

Shelf life of the product is two years.

After opening it is recommended the product should be used within three months.

Store away from direct sunlight.

Only use the product in the intact container.

Keep out of reach of children.

Manufacturer:

ENIKAM d.o.o.

C. Andreja Bitenca 118

1000 Ljubljana, Slovenia

info@enikam-oxy.com

Tel.: +386 31 634 900

www.enikam.com, www.oroxid.com

Date of last revision: 03/2023 - V2.0

Description of the symbols on the packaging:

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