AQUACEL™ EXTRA™
Hydrofiber™ Dressing with Strengthening Fibre / Hydrofiber™ Verband mit Verstärkenden Fasern / Medicazione in Hydrofiber™ con Fibra Rinforzante / Pansement Hydrofiber™ avec Renfort Fibre / Apósito de Hydrofiber™ con fibra reforzante / Penso Hydrofiber™ com Fibra de Reforço / Hydrofiber™ Verband mit Verstärkenden Vezels / Hydrofiber™ Förband med Förstärkande fiber / Kuituvahvistettu Hydrofiber™-haavasiidos / Hydrofiber™ Bandage med forstärkade fibrer / Hydrofiber™ Bandasje med Forsterkede Fibrer / Εμπίκουρος Hydrofiber™ με ενυωγούσες ίνες / مع تسجيح مقوى Hydrofiber™
INSTRUCTIONS FOR USE / GEBRAUCHSANLEITUNG / ISTRUZIONI PER L'USO / CONSEILS D'UTILISATION / INSTRUCCIONES DE USO / INSTRUCIONES DE USO / GEBRUIKSIJNSTRUCTIES / BRUKSANVISNING / KÄYTTÖOHJEET / BRUGSANVISNING / BRUKSANVISNING / OAHITSE XPHIHE / تعليمات الإستعمال
ConvaTec II
ENGLISH

PRODUCT DESCRIPTION

AQUACEL™ EXTRA™ Hydrofiber™ Dressing with Strengthening Fibre is a soft, sterile, non-woven pad dressing composed of sodi-um carboxymethylcellulose and regenerated cellulose fibre for strengthening. This conformable and highly absorbent dressing absorbs wound fluid and transforms into a soft gel, which maintains a moist environment to support the body's healing process and aid in the removal of nonviable tissue from the wound (autolytic debridement), without damaging newly formed tissue.

INDICATIONS

Under medical supervision AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre may be used for the management of:

- leg ulcers, pressure ulcers (Stage II-IV) and diabetic ulcers. surgical wounds (e.g., post-operative, wounds left to heal by secondary intent and donor sites),
- partial thickness burns.
- traumatic wounds (e.g., abrasions and lacerations).
- exudate absorption in oncology wounds (eg., fungating cuta-neous tumours, cutaneous metastases and Kaposi's sarcomas).

CONTRAINDICATIONS

AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre should not be used on individuals who are sensitive to or who have had an allergic reaction to the dressing or its components.

PRECAUTIONS AND OBSERVATIONS

- Caution:** Sterility is guaranteed unless pouch is damaged or opened prior to use.
- This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination. Physical properties of the device may no longer be opti-mal for intended use.
- Appropriate supportive measures should be taken where indi-cated (e.g., use of graduated compression bandaging in the management of venous leg ulcers or pressure relief measures in the management of pressure ulcers/sores).
- The control of blood glucose, as well as appropriate supportive measures, should be provided with diabetic foot ulcers.
- Infection is not a contraindication to the use of AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre. Should infection develop during the use of the dressing, antibiotic therapy should be initiated, as clinically indicated, by a health care professional.
- AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre can facilitate the control of minor bleeding.
- If removing the dressing is difficult, the dressing should be fully saturated with sterile saline or water and removed slowly.
- Because AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre provides a moist environment that sup-ports the growth of new blood vessels, occasionally the delicate newly formed blood vessels may produce a blood stained wound fluid.
- AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre is not intended for use as a surgical sponge.
- AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre is not intended for use within internal body cavities or within closed wounds

DIRECTIONS FOR USE

- Exuding Wounds (Excluding Partial Thickness Burns)**
 - Before applying the dressing, cleanse the wound area with an appropriate wound cleanser.
 - AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre should overlap 1cm onto the skin surrounding the wound.
 - Apply the dressing to the wound and cover with a moisture retentive dressing (e.g., DuoDERM™ Extra Thin or Versiva® XC), or other appropriate secondary dressing. See relevant individual package inserts for complete instructions for use.
 - All wounds should be inspected frequently. Remove the AQUA-CEL EXTRA Hydrofiber Dressing with Strengthening Fibre when clinically indicated (i.e., leakage, excessive bleeding, or suspi-cion of infection).
 - AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre can be left in place for up to 7 days, where clinically indicated, (except in burns and donor sites where it can be left in place for up to 14 days).

2. Dry Wounds

- In addition to the directions for use set forth above:
- Place the AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre on the wound and wet with sterile water or saline over the wound area only.
- Cover the dressing with a moisture retentive dressing such as DuoDERM™ Extra Thin to avoid drying out of the dressing and subsequent dressing adherence to the wound.

3. Partial thickness burns

- Before applying the dressing, cleanse the wound area with an appropriate wound cleanser.
- The AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre should overlap at least 5cm onto the skin surrounding the burn or other adjacent AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre, as the dressing will shrink as it absorbs the exudate.
- The AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre should be covered with a sterile loosely woven gauze pad and appropriately secured. In the immediate post burn period (up to 4 days) large volumes of wound exudate may require that the saturated dressing be removed and replaced with new AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre.
- Remove the cover periodically and inspect the AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre while it remains in place on the burn.
- In this indication adherence to the wound bed of the AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre is a desired characteristic. Non adherence of areas of the dress-ing may indicate deepening of the wound or infection. Areas of the dressing may be cut away to facilitate assessment. The exposed areas should then be treated appropriately.
- As the burn wound re-epithelialises, the AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre will detach or be easily removed.
- For partial thickness burns and donor sites, AQUACEL EXTRA Hydrofiber Dressing with Strengthening Fibre may be left in place for up to 14 days or until clinically indicated.
- If the burn is infected frequent inspection of the wound may be necessary.

Discard any unused portion of the dressing

Store in cool dry place. (10°C - 25°C/50°F – 77°F).

If further information or guidance is needed, please contact ConvaTec Professional Services

© 2016 ConvaTec Inc.
*/™ indicates a trademark of ConvaTec Inc.

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*/™ Indica marca de ConvaTec Inc.