## ENGLISH PACKAGE CONTENTS

The **ZuhauseTEST Fruchtwasser (Amniotic Fluid)** contains:  
- 5 test panty liners, and one instruction for use

Keep the **ZuhauseTEST Fruchtwasser (Amniotic Fluid)** sealed and use it immediately after opening.

## CONDITIONS FOR USE

To ensure valid results, do not use the **ZuhauseTEST Fruchtwasser (Amniotic Fluid)** device if the answer to any of the following questions is YES:

- 1) Is it less than 12 hours after sexual intercourse?
- 2) Is it less than 12 hours after vaginal douching?

During bleeding results reading may need professional assistance. Report all bleeding situations to your physician.

Disregard positive result and retest if there is a possibility of wetness/ wetness remains from shower, bath etc, as these may interfere with the results.

In case of sensitization or irritation of skin, at the application site, stop use and contact your physician.

## DIRECTIONS FOR USE

- 1) Tap water can interfere with the test – wash and dry your hands.

- 2) Unwrap the **ZuhauseTEST Fruchtwasser (Amniotic Fluid)** panty liner. The panty liner contains a pale yellow indicator.


- 3) Attach the panty liner to your underwear. The pale yellow indicator strip should sit directly against your vagina. Wear the panty liner as you would a normal panty liner until you feel wetness (do not exceed 12 hours).

**Note:** use the panty liner while sitting, standing or walking. Do not use the panty line while lying down.

- 4) When you feel a fluid leak, remove the panty liner and wait 15 minutes. Then check for a color change on the panty liner's yellow background. Also when replacing with a new panty liner, check for a color change.

- 5) If you feel wetness and the panty liner is completely yellow, after 15 minutes, consider using another test.

- 6) Dispose the panty liner as you would have disposed other hygiene products.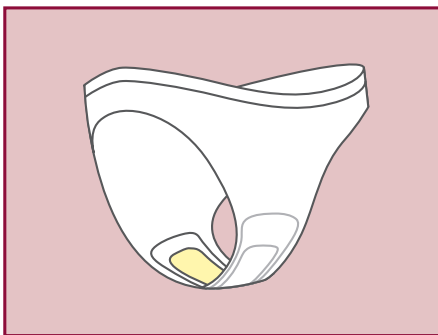

## RESULTS INTERPRETATION

**15 MINUTES AFTER REMOVAL LOOK FOR A COLOR CHANGE ON THE PANTY LINER.**


Result reading should be done in a well illuminated place.

15 min.

## PROBABLE AMNIOTIC FLUID LEAK

If there is a color change, partial or complete, to blue or green (any intensity), in any size, shape or location, on the panty liner, the fluid leak is probably amniotic fluid. The color change can look cloudy and can be more intense along part of the yellow strip edge. In any such case, consult your physician or go to the hospital without delay.

**Important:** The indicator strip may also turn blue or green if vaginal infection is detected. If the indicator strip turns blue or green, always consult your physician.

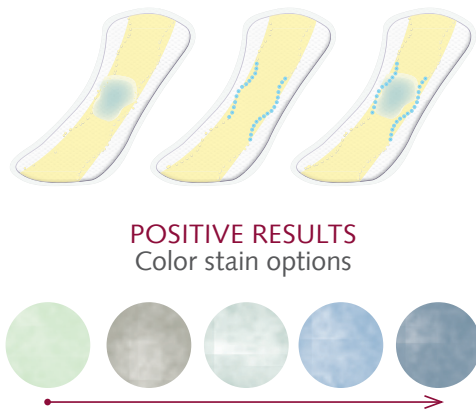
**POSITIVE RESULTS**  
Color stain options


## PROBABLE NEGATIVE RESULT

If the panty liner did not change color (remained yellow or turned yellow after 15 minutes), the vaginal wetness was probably not caused by amniotic fluid. If you continue to experience vaginal wetness, apply a new panty liner and repeat the test and consult your physician.

**Warning:** In case of heavy leaks or the suspect of broken waters, please seek medical advice immediately, even if the device does not change color.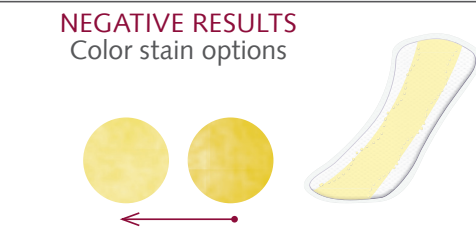

**NEGATIVE RESULTS**  
Color stain options

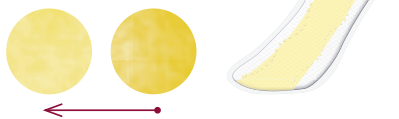## PERFORMANCE DATA

A total of 232 pregnant women were enrolled in a pivotal study of the **ZuhauseTEST Fruchtwasser (Amniotic Fluid)** Kit. The **ZuhauseTEST Fruchtwasser (Amniotic Fluid)**, as a continuance test, demonstrated a sensitivity of 97.06% (99/102), with a 95% CI of [91.64%;99.39%] and specificity of 96.92% (126/130), with a 95% CI of [92.31%;99.16%].

- 1) Bornstein J, Geva A, Salt I, Fait V, Schoenfeld A, Shoham HK, Sobel J. Noninvasive diagnosis of premature ruptured amniotic membranes using a novel polymer. Am J Perinatol. 2006 Aug; 23(6):351-4.
- 2) Bornstein J, Ohel G, Sorokin Y, Reape KZ, Shnaider O, Kessary-Shoham H, Ophir E. Effectiveness of a novel home-based testing device for the detection of rupture of membranes. Am J Perinatol. 2009 Jan;26(1):45-50.

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